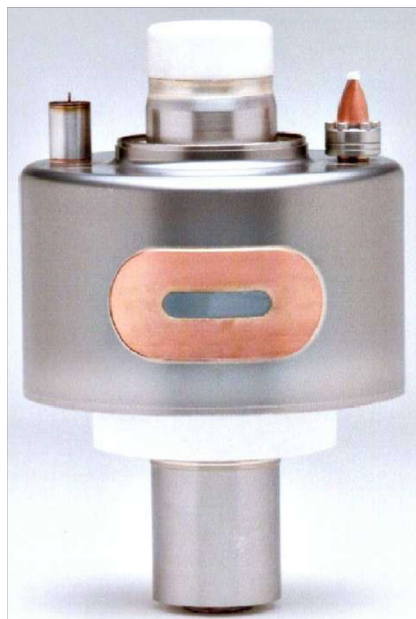
Technical Data

DU 6308 X-ray Tube

453570030300

453575069168 REV. A

DUNLEE

X-ray Tube Technical Data
4.7 MJ/6.3 MHU CT Insert
0.7 x 0.6/0.9 x 0.9 Focal Spots

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This X-ray Tube will be used as part of a registered and controlled X-Ray Tube Housing Assembly. Please refer to corresponding X-Ray Tube Housing Assembly document for important safety warnings, servicing and operating instructions, warranty and return procedures associated with installation, use and maintenance of this product.

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Copyright

© 2012 DUNLEE

All rights reserved. Reproduction in whole or in part is prohibited without the prior written consent of the copyright-holder. DUNLEE reserves the right to make changes in specifications or to discontinue any product, at any time without notice or obligation, and is not liable for any consequences resulting from the use of this publication.

Equipment is subject to change without notice. All changes will be in compliance with regulations governing manufacture of medical equipment. Printed in the USA. Document originally drafted in English.

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DISPOSAL

Dunlee Manufactures state-of-the-art x-ray equipment in terms of safety and environmental protection. Under normal operation, there are no risks to persons or the environment. To comply with regulations, it is necessary to use materials that may be harmful to the environment and therefore must be disposed of in a proper manner. Contact your Dunlee sales and service organization for proper handling and disposal.

This metal x-ray tube may contain beryllium. Disposal of this product must be through appropriate channels. (This statement is valid only for metal x-ray tubes, please disregard for glass tubes).

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ENVIRONMENTAL STATEMENT

WE STRIVE TO BE ENVIRONMENTALLY CONSCIOUS WITH THE PRODUCTS AND THE MATERIAL RETURNED FOR DISPOSAL. CONSEQUENTLY, DUNLEE CAREFULLY DISASSEMBLES AND PROCESSES PARTS AND MATERIALS ACCORDING TO EARTH FRIENDLY OBJECTIVES. SOME OF THESE MATERIALS MAY BE USED IN THE MANUFACTURE OF NEW PARTS AND ASSEMBLIES AS PART OF THIS GREEN INITIATIVE. ALL DUNLEE PRODUCTS WILL ACHIEVE THE SAME HIGH LEVEL OF QUALITY AND FUNCTIONALITY OUR CUSTOMERS HAVE COME TO EXPECT.

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INSTALLATION AND ASSEMBLY

The X-ray Tube must be installed into an appropriate x-ray tube housing assembly for safe and proper use. Only trained and qualified personnel must perform this final assembly and be performed in accordance with the documentation and guidelines provided by Dunlee. Failure to follow the appropriate procedure for final assembly will void the warranty.

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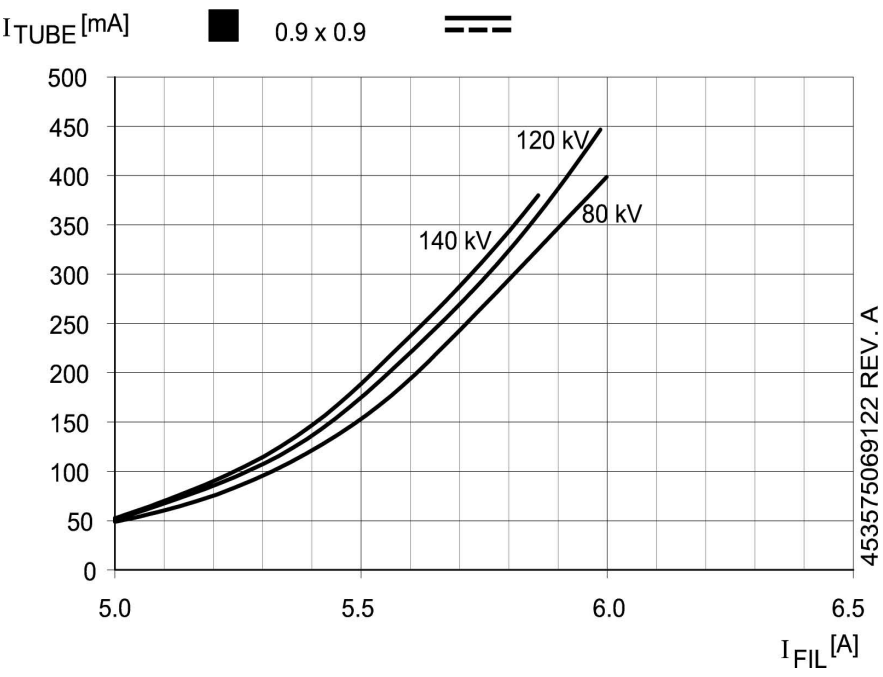
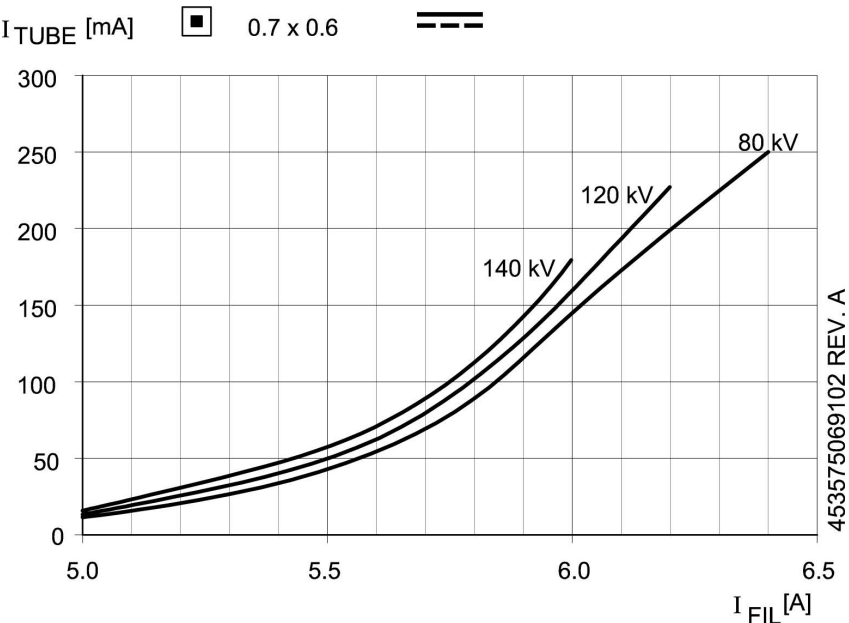
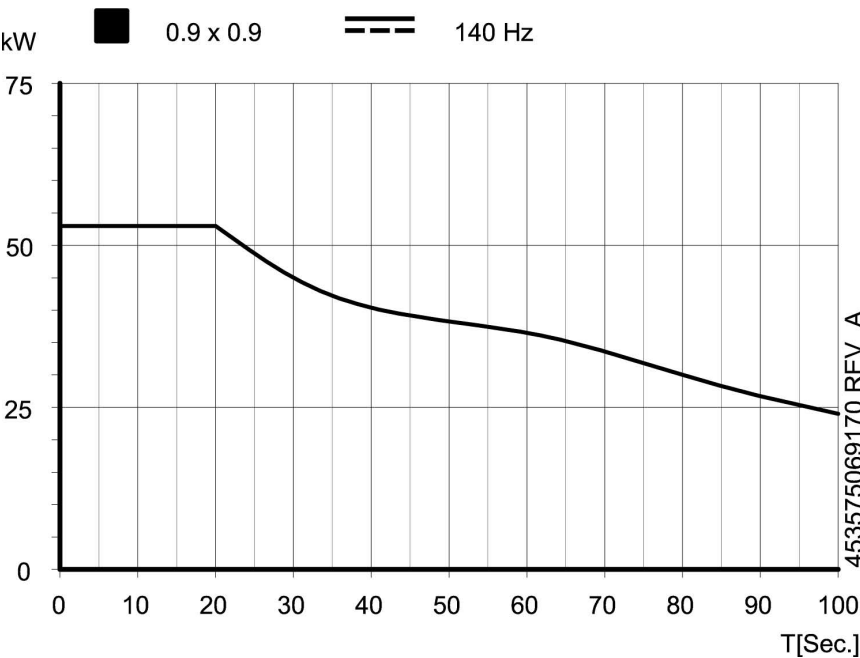
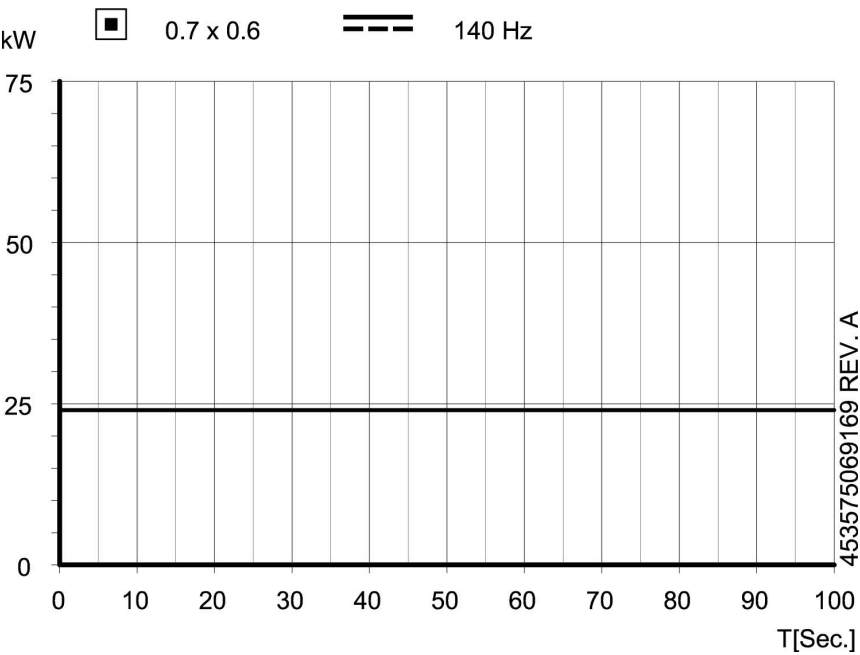
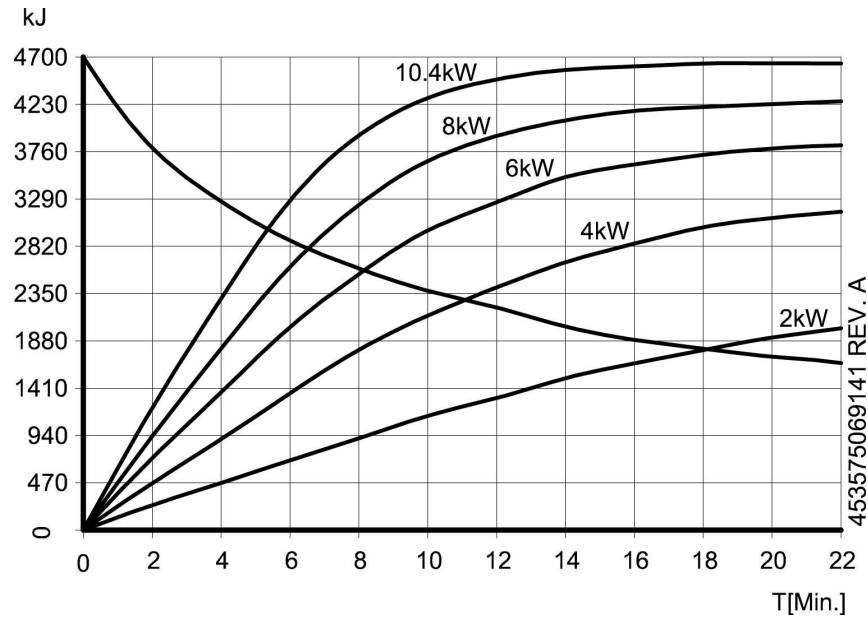
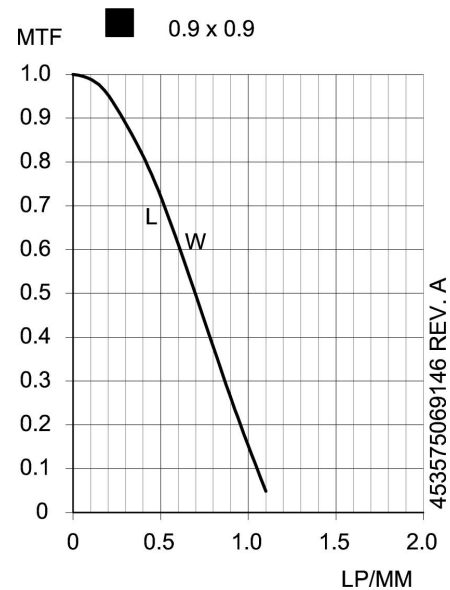
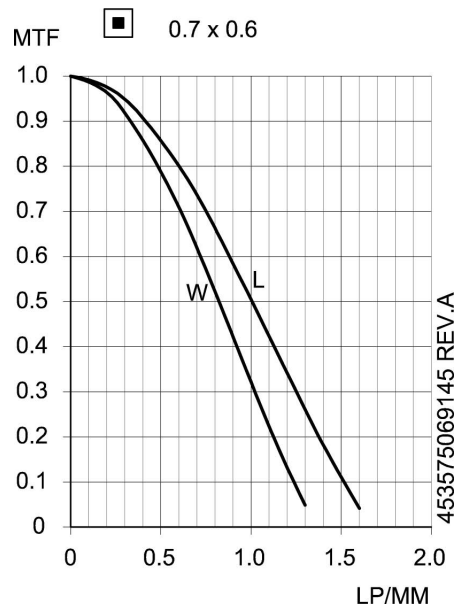
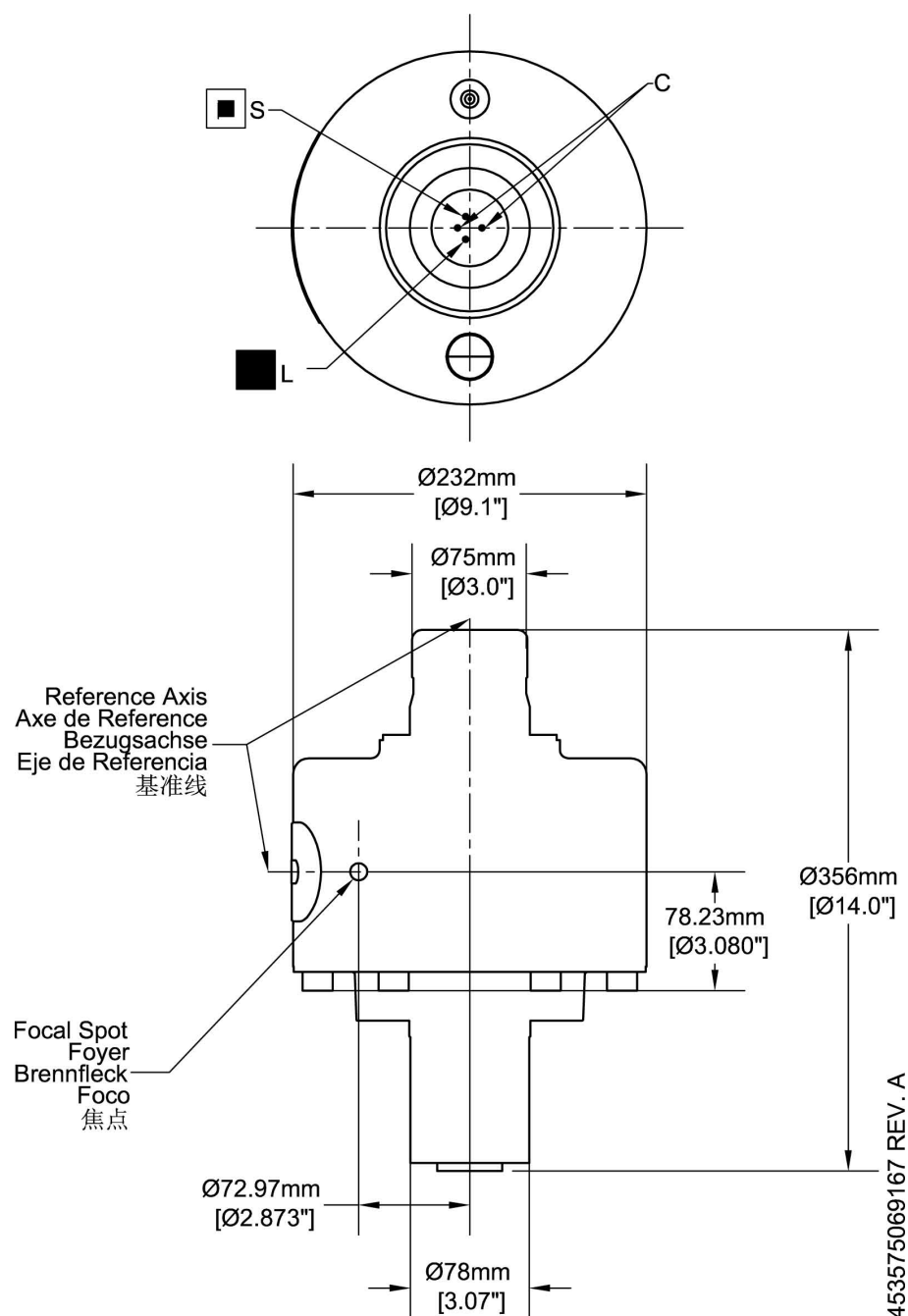
Section 8 (Dunlee):

Service and Manufacturing Locations

Dunlee World Headquarters

555 North Commerce Street
Aurora, Illinois 60504 USA
Tel: +1 800.238.3780 (U.S. & Canada)
+1 630.585.2100
Fax: +1 630.585.2125

Arlington Facility

2312 Avenue J
Arlington, Texas 76006 USA
Tel: +1 800.238.3780
(U.S. & Canada)
+1 817.640.7666
Fax: +1 817.640.6644

European Union Representative

Philips Medical Systems
Dept: Corporate Quality & Regulatory
Venpluis 4
5684 PC Best
The Netherlands

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Philips Healthcare (Suzhou) Co., Ltd.